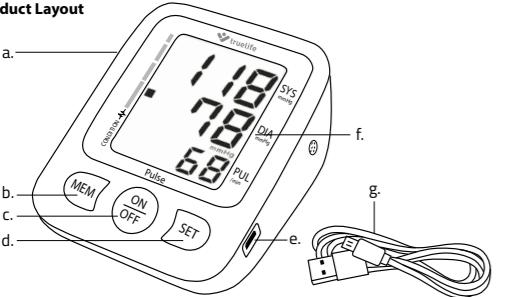
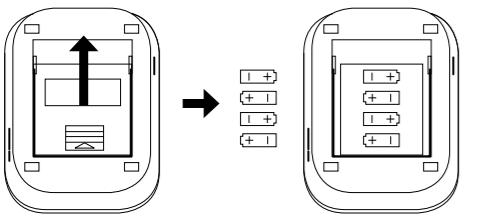
1. Product Layout

2. Battery Installation

EN – INSTRUCTIONS FOR USE

Before first use, the consumer must first read the instructions and safety precautions of the product.

1. Product Layout

- a. Cuff tube inlet
- b. Reading memory button
- c. On / Off button
- d. SET button
- e. USB power input
- f. Display
- g. USB cable

2. Battery Installation

Remove the battery cover on the back of the blood pressure monitor and insert 4 AA 1.5V alkaline batteries in the direction as shown by the arrow. Make sure the batteries are inserted in the correct direction of polarity as shown in the battery compartment. Do not use batteries after the expiry date. The batteries must be replaced if the battery symbol appears on the display when the pressure monitor is turned on. The symbol indicates low battery power. At critically low battery power levels, the device will not take readings. The device is not suitable for use with rechargeable batteries.

If you do not intend to use the device for an extended period of time, remove the batteries.

3. Using the Adapter


Do not connect the adapter while the device is turned on. When connected to an adapter, the device does not draw any battery power. If you disconnect the adapter from the device whilst taking a reading, you must plug the adapter back in again and repeat the reading.

4. Preparing to Take a Reading

- Before taking a reading, rest for 5–10 minutes, avoid eating, drinking, smoking, exercising and bathing.
- Connect the cuff hose to the body of the blood pressure monitor.
- Take off your outer wear. Keep on only thin clothing or remove all clothing from your arm.
- Put the sleeve of the cuff on your arm to a level above your elbow. Always measure on the same arm (ideally the left one).
- As the pressure changes during the day (usually the highest in the morning), take measurements at the same time each day.
- Repeated measurements accumulate blood in the arm and may cause measurement errors. The interval between measurements should be at least 1 minute. You can shorten the interval by raising your arm so that the blood drains.
- Any attempt to support your arm whilst taking a reading may increase the pressure and distort results. Any activity, including sleeping, affects the measurement results.
- The arm on which you take the reading should be kept so that the artery is roughly level with the heart. The measurement result will be more accurate.
- Place the cuff on the left arm with the tubing pointing downwards and the bottom of the cuff approximately 2–3 cm above the elbow.
- Make sure that the cuff is not tightened too tightly. You should be able to insert two fingers between the cuff and the arm.
- Place your hand upside down on the table so that you feel comfortable with the cuff roughly at heart level. Do not bend the tubing.
- If it is not possible to take a reading on the left arm, use the right arm instead. However, the reading should be taken on one arm only.

5. Taking a Reading

- Press the START / STOP button. All the symbols appear on the display. Then the display shows 0 and the device starts inflating the cuff. You can follow the cuff pressure on the display.
- Once the required cuff pressure has been reached, the pump stops inflating the cuff and the pressure gradually decreases.
- When the device detects a signal, a flashing heart symbol appears on the display. The symbol flashes once each time the heart beats.
- When the measurement is complete, a beep sounds and the display shows systolic and diastolic pressure and heart rate.
- The measurement results will remain on the display until the device is turned off. If idle, the device will turn itself off within 3 minutes. If the monitor does not measure the results correctly, the display will show the "E" icon.
- If for any reason it is necessary to interrupt the measurement (e.g. due to nausea), press the START / STOP button. The device immediately relieves the cuff pressure automatically.

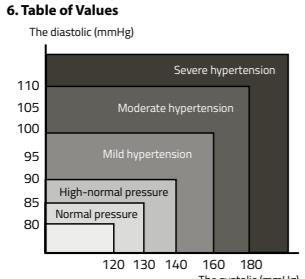6. Table of Values

7. Error indicator

- a) Symbol E-1**
You may have an incorrectly attached cuff – Adjust it to the correct position and repeat the measurement.
- b) Symbol E-4**
Cuff hose is improperly inserted – Check tubing connection and adjust or reconnect if necessary. Then repeat the measurement.

Monitor or arm have moved whilst taking a reading – Keep your body and the monitor motionless whilst taking a reading. Repeat measurement.

The systolic pressure less than 30 mmHg – Manually increase the pressure for the desired result. Repeat the measurement.

c) The E-5

Pressure is greater than 300 mmHg – The device is releasing air itself.

d) Low Battery Symbol

Replace discharged batteries with new ones and repeat reading.

e) Arm symbol in motion

The symbol appears when the hand is moving excessively.

f) Heart Symbol

The symbol appears when the heart beats irregularly.

Measurement Method

Oscillometric

Pressure Measurement Range

0–300 mmHg

Heart Rate Measurement Range

40–180 bpm

Pressure Measurement Accuracy

± 3 mmHg

Heart Rate Measurement Accuracy

± 5%

Memory

60 readings

Power Supply

4x AA Battery DC 6V or AC Adapter

AutoOff

After 3 Minutes

Pressure Monitor Weight

300 g (without cuff and batteries)

Pressure Monitor Dimensions

120 x 93 x 59 mm (without cuff)

Lifetime

approx. 10,000 readings

Operating Temperature

5–40 °C

Operating Humidity

10–90% RH

8. Maintenance

- After use, store the blood pressure monitor in its packaging and protect it from damage.

- Store the blood pressure monitor in a dry environment.

- Keep the blood pressure monitor clean. Clean it with a dry cloth. Do not use abrasive or corrosive cleaners.

- Neither the blood pressure monitor nor any of the components should come into contact with water.

- Make sure the blood pressure monitor is off before cleaning. For cleaning it is possible to use distilled water with a 10 % bleach solution.

- Use a spray, moisten a soft cloth with bleach or detergent until it is soaked.

- Squeeze excess moisture from the fabric to prevent the unit from dripping or cuff getting wet.

- Wipe the entire surface of the cuff thoroughly. Make sure you clean the inside and outside of the cuff properly. Be careful not to let any moisture get into the blood pressure monitor.

- Thoroughly wipe all parts of the cuff with a dry cloth. Unfold the cuff and allow it to air dry.

- Do not clean the blood pressure monitor or the cuff with diesel, thinner or petrol.

- Do not expose the device to extreme heat or cold, moisture, and direct sunlight.

- If you will not be using the device for more than 3 months, remove the batteries.

- Elemt s.r.o. cannot be held responsible for any damage caused by insufficient maintenance and non-observance of the instructions in the manual.

9. Precautions

- Use only the original charger supplied as an optional accessory to the blood pressure monitor.

- Do not use the charger in a high humidity environment. Never touch the charger with wet hands or while standing in water.

- When supplying power, leave enough space around the charger for air circulation. Do not cover the charger with paper or other objects that could lead to poor cooling. Do not use the charger stored in the transport packaging.

- Do not use the charger or device if it is obviously damaged. If the device is damaged, do not repair it yourself.

- In case of excessive heating, switch off the appliance immediately and disconnect it from the power supply.

- Charge the device only under supervision.

Warning: This product is intended to be used in households and similar applications.

Import:

elemē s.r.o. Bráškovská 15, 16100 Prague 6, Czech Republic. Made in P.R.C.

Well Kang Limited, The Black Church, St. Mary's Place, Dublin 7, D07 P4AX, Ireland

Xiamen Ants-Bro Technology Co., Ltd, 4F, 5th Building, Technology business establishing center, No. 289 Wengjiao Road, Haicang District, Xiamen City, Fujian Province, China

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EC REP Well Kang Limited, The Black Church,